

Prepare-on-Site (90 Ready to use wipes)



Long lasting, economical & easy to use disinfection wipes



- Choose your wipe material
- Choose your cleaning solution or disinfectant
- Create your ready-to-use wipes

TouchPoint Disposable POS (Prepare-on-Site) wipes allow you to create your own pre-moistened cleaning wipes with your own cleaning or disinfection solution.

The roll of 90 wipes are folded in such a way to ensure that the wipes are moistened evenly with the correct amount of solution.

This is the most economical disposable cleaning system allowing you the flexibility to use your preferred cleaning or disinfecting solution.

A new Ready-to-Use wipe is used to clean each room reducing the possibility of cross contamination through the cleaning process.

The bucket keeps the wipes ready to use, is washable and can be disinfected after use.

Each roll contains 90, 28 X 32 cm disposable Wipes specifically structured to deliver disinfectant solution to environmental surfaces.

Dispenser bucket and lid can be disinfected and reused. The lid seals in moisture keeping your disinfectant solution and wipes away from the elements.

Ideal for creating 1000ppm available chlorine disinfectant solution wipes to manage infectious cleans in Australian Hospitals and Aged Care facilities.

Simply add 2 Chloreside tabs, 2 litres of warm water, a roll of Disposable Disinfection Wipes and you're ready to disinfect.

Note: Your Disposable Chloreside Disinfectant Wipes contain disinfectant and surfactant and are suitable for cleaning and disinfection.



How to use:

1. Prepare your Chloreside Disinfection Wipes
2. Pre-clean all surfaces starting with TouchPoints, with your disposable disinfection wipes. Use as many wipes as you need to.
3. Disinfect all surfaces using new wipes from your disposable disinfection bucket using again, as many wipes as required.

By pre-cleaning, followed by disinfection application you will be effectively performing a two step process using 1000PPM (or other strength) chlorine disinfectant. This two steps should allow your chlorine solution to be active on the surface for the required 10 minutes.

Preparing your Disposable Disinfection Wipes



1. Place a new roll of Disposable Disinfection wipes into a clean bucket.



2. Pour 2 litres of disinfection solution onto the wipes.



3. Pull the first wipe through the opening, lock the lid by placing onto bucket and rotating to the right.



4. Press lid firmly onto bucket to keep contents fresh.



5. Close the lid by rotating the seal shut keeping the first wipe ready for use.



6. Place bucket on trolley ready for use.